


## INSTRUCTIONS FOR COLLECTION

### Autism Spectrum Disorder - Genetic Test Panel

Prior to collecting the saliva sample be sure the patient has not had anything to eat or drink, in the last 30 minutes and has not brushed their teeth in the last 60 minutes.


#### STEP 1



**Open sealed package**

Contents include one oral swab with reversible cap - the liquid in the genetic swab will be purple/blue

#### STEP 2




**Collect sample**

Place swab tip along the outer, lower gum line and rub back and forth

Firmly rub 10x each side and avoid touching the teeth


#### STEP 3

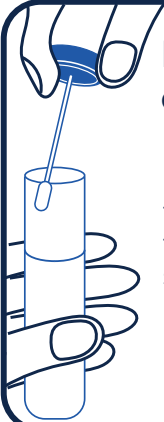

**Remove cap**

Hold the swab upright, unscrew the cap, do not touch the swab or spill the liquid

#### STEP 4

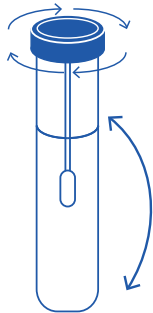


**Place swab in tube**

Tightly seal the cap and then invert the tube 15 times

Turn the tube upside down 15x

#### STEP 5

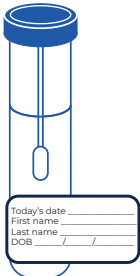

**Label the tube**

Label with patient's full legal name and date of birth, do not cover the barcode

Today's date \_\_\_\_\_  
First name \_\_\_\_\_  
Last name \_\_\_\_\_  
DOB \_\_\_\_\_ / \_\_\_\_\_ / \_\_\_\_\_